

UNIC[®] Revision

Surgical Technique



Revisions

Removal of UNIC® primary components p.3

Implantation of a humeral revision stem p.4

Glenoid revision p.7

SNAPSHOT OF THE INSTRUMENTATION SET p.9

IMPLANTS REFERENCE LIST p.12

Removal of UNIC® primary components


Humeral head removal:

Place the humeral head extractor between the head and the stem ❶

Do not lever.


Impact the extractor so that the wedge moves into the space between the head and the stem.

Module extractor	E28 015
Broach and implant holder	E28 028
Slap hammer	H01 015
Humeral head extractor	E28 128
M6 impaction shaft	S01 026
Glenoid sphere extractor	E28 211

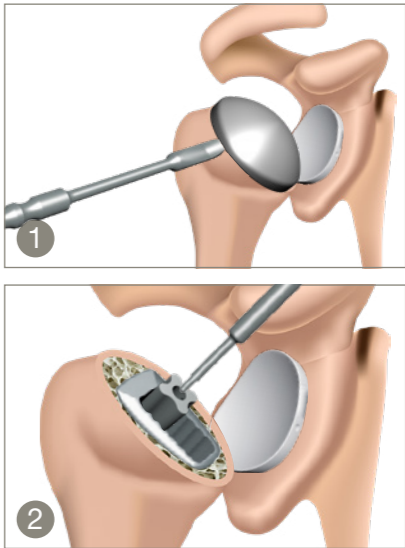

Removal of the connection modules:

Thread the module extractor through the hole in the module ❷, all the way until it comes into contact with the humeral stem. Continue the screwing motion until the module is pulled out of the stem.

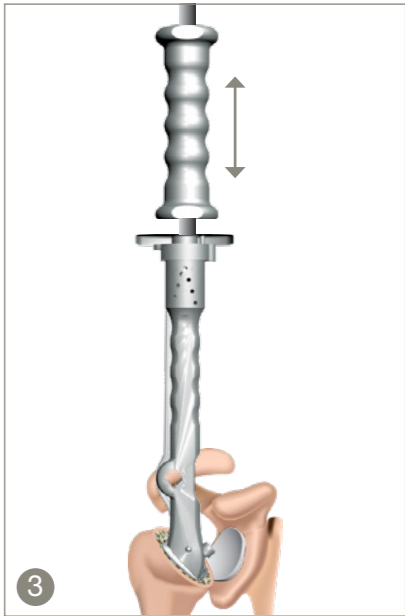


Removal of the humeral stem:

After having removed the connection module, place the broach and implant holder in the stem and lock it. The slap hammer can be screwed onto the stem holder and used to extract the stem ❸.

Removal of the glenoid sphere:

Unscrew the security screw of the glenoid sphere ❹.

Screw the glenoid sphere extractor into the glenoid sphere until it pulls out ❺.

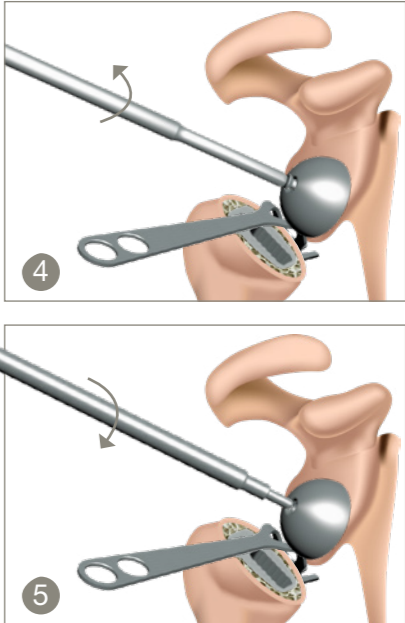

WARNING





In cases of removal of a ceramic glenosphere use the specific extractor E28 214.

Implantation of a Revision Humeral stem

INFOS

Implantation of a revision stem requires use of the Reverse Shoulder set of instruments (E28 9105) plus the revision Shoulder set of instruments (E28 9102).

Size 0 revision broach	E28 R000
Size 1 revision broach	E28 R001
Size 2 revision broach	E28 R002
Size 3 revision broach	E28 R003
Size 4 revision broach	E28 R004
Broach and implant handle	E28 028
Securing fixation screw	E28 019
Distal targeting jig	E28 020
Crossbar	E28 023
M7 yellow locknut	S01 038
M6 T shaped locknut	S01 029
Large sleeve guide	H38 052
4.5mm - 132mm long drill guide	H38 060
4.5mm drill, length 195mm	H38 054
Locking pin depth gauge	H38 055
Screw holder	S01 011

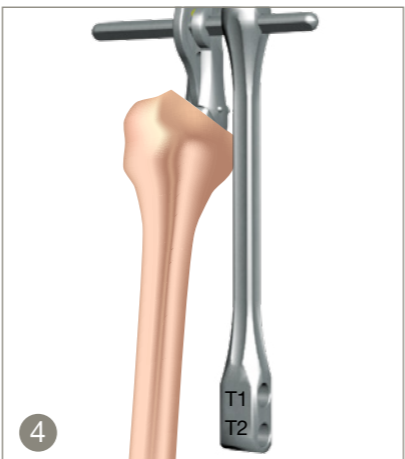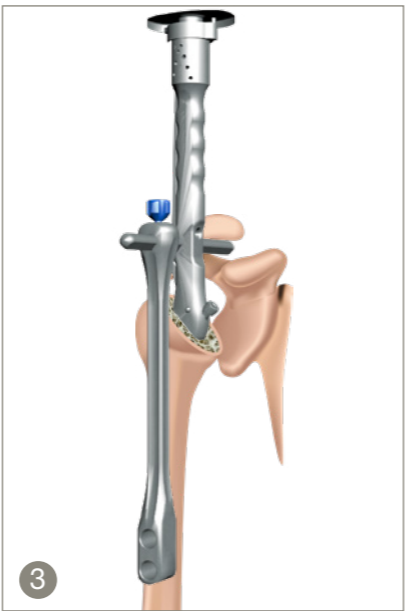

The procedure is identical for a cementless HA coated humeral stem, and for a cemented shiny-polished humeral stem.

After removal of the implants previously in situ, place the 0 size revision broach on the broach and implant handle. Start broaching and incrementally increase broach size until correct medullary fill, stability and rotation are achieved ①.

During broaching and when implanting the definitive stem, control of the retroversion can be made using the E28 007 orientation guide (see page 8 of Anatomic or Reverse UNIC primary surgical technique).

Implant the definitive stem chosen using the broach and implant handle.

When implanting a cemented humeral stem, introduce first a cement restrictor, then the bone cement with a syringe in a retrograde manner. Next, introduce the definitive stem until final seating, and leave still until the bone cement is fully set.

Distal locking of the locked revision stem:

Leave the stem holder on the stem.

Stability can be augmented by using the fixation screw securing the handle and stem together. ② Attach the distal targeting jig on the crossbar.

- Check orientation as follows ③ :
- The distal jig is reversible.
 - One way for stem sizes 1 and 2.
 - The other way around for size 3 and 4.
 - The size required should face laterally ④.

WARNING

Size 0 cannot be distally fixed as the diameter is too narrow.

Slide the crossbar with jig into the handle as close to the humerus as possible.

Lock it with the yellow M7 locknut. ⑤

Lock the distal jig onto the crossbar with the M6 locknut. Introduce the large sleeve guide into the most proximal hole of the distal jig and push it until it comes into contact with the humeral cortex ⑥.

Slide the 4.5mm - 132mm long drill guide into the large sleeve guide. ⑦ Mount the 4.5mm drill, length 195mm on a power tool and drill through both first and second cortex ⑧. The graduations on the drill allow for instant reading of drill depth.

Remove the drill and drill guide.

Check the length of the locking pin with the depth gauge ⑨.

Select the appropriate length of locking pin.








Put the screw holder onto the 3.5mm screwdriver.

Place the hexagonal part of the screwdriver into the locking pin head, and screw the screw holder

over the thread on the locking pin head. The locking pin is now held firmly by the screwdriver and holder.

Slide both into the large sleeve guide previously left in place ⑩.

Screw the locking pin into the humerus but not tight, and then withdraw the screwdriver leaving the locking pin and holder in place. This gives more stability whilst preparing the most distal pin ⑪.

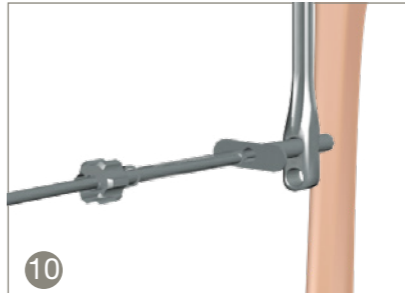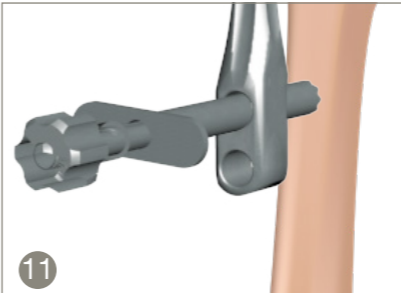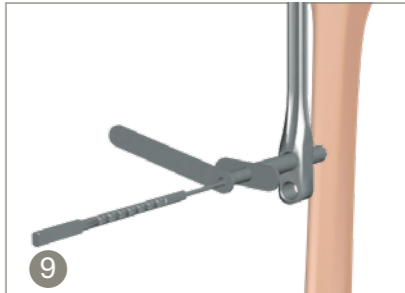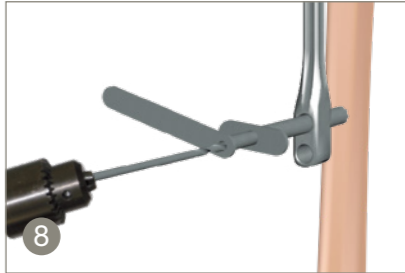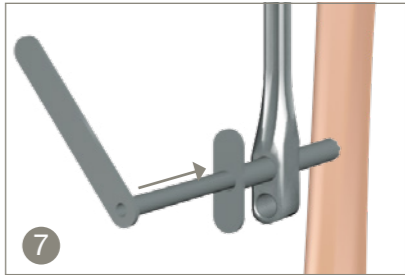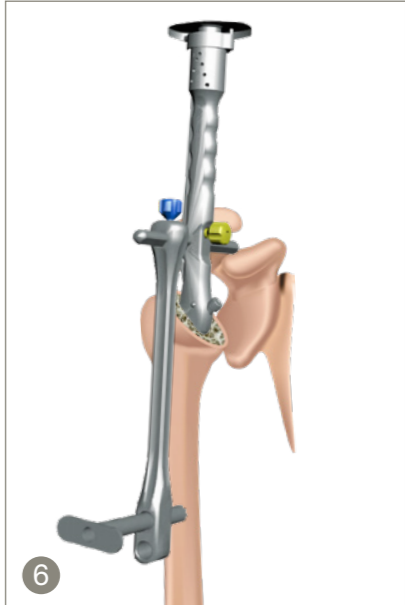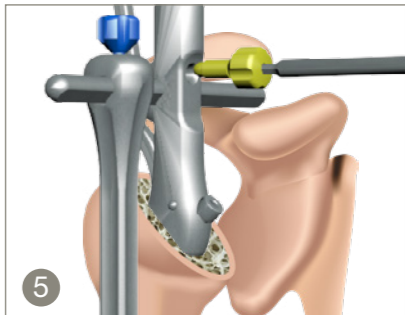

Introduce the 2nd large sleeve guide into the most distal hole of the distal jig and push it until it comes into contact with the humeral cortex 12.

Slide the 4.5mm - 132mm long drill guide into the large sleeve guide 13. Mount the 4.5mm drill, length 195mm on a powertool and drill through both first and second cortex. The graduations on the drill allow for instant reading of drill depth 14.

Remove the drill and drill guide.




Check the length of the locking pin with the depth gauge 15.

Select the appropriate length locking pin. Mount it onto the screwdriver and holder and screw it into place until firm cortical contact 16.

Remove the screwdriver and large sleeve guide.

Remove the locking pin holder from the proximal large sleeve guide, taking care to unscrew it from the locking pin head.

Finish tightening the proximal locking pin up to firm cortical contact. Remove the holder, large sleeve guide and the jig 17.

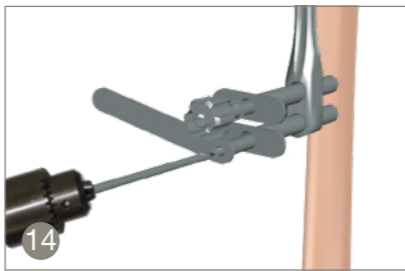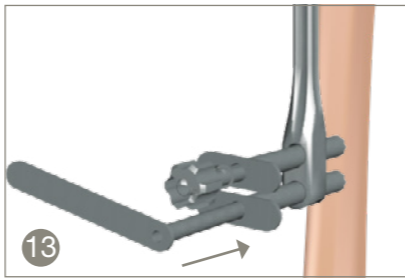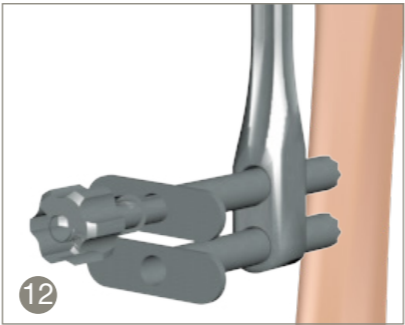

Glenoid revision

INFOS



Implantation of a revision glenoid requires use of the Reverse Shoulder set of instruments (E28 9105) plus the Glenoid revision set of instruments (E28 9103).

Assemble the glenoid guide pin sleeve on the Revision glenoid trial of the appropriate side and onto the M6 impaction shaft 1.

Position the guide over the patients' glenoid 2.

Mount a 2.5mm pin length 150mm onto a powertool and drill it through the guide pin sleeve into the bone 3.

Remove the assembly leaving the guide pin in place 4.

If necessary the concavity of the glenoid can be reamed using the 30mm reamer.

Mount the conical drill for revision peg onto a powertool.







Glenoid guide pin sleeve	E28 236
Revision glenoid trial right	S01 026
Revision glenoid trial left	E28 238
M6 impaction shaft	E28 239
2.5mm pin length 150mm	E28 150
30mm glenoid reamer	E28 121
Conical drill for revision peg	E28 234
Back and forth reamer	E28 009
Reamer shaft	E28 120
T handle	E28 235
20mm trial revision peg	E28 242
25mm trial revision peg	E28 243
30mm trial revision peg	E28 244
Helical baseplate impactor	E28 210
Head impaction endpiece	E28 108
2.5mm, 35mm drill	S01 040
Flexible AO attachment	S01 033
3.2mm guide	E28 232
Depth gauge	6006-0071

Place the conical drill over the guide pin and drill the glenoid 5.

The drill can be either drilled to a depth of 20mm, 25mm or 30mm: 6

- 20mm stopping at the end of the flutes.
- 25mm stopping at the groove
- 30mm stopping at the stop.

Remove the conical drill and guide wire.

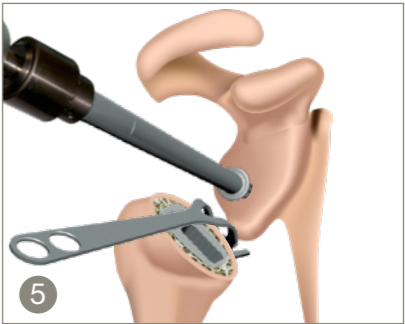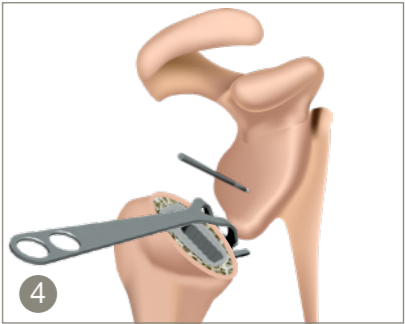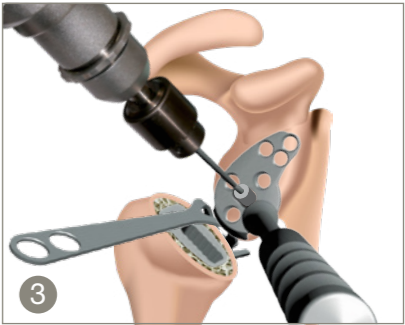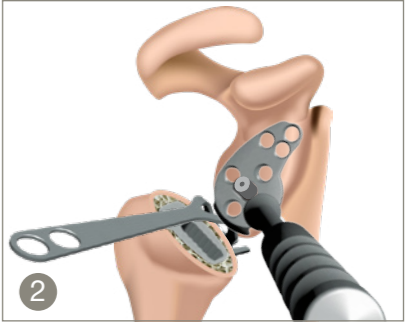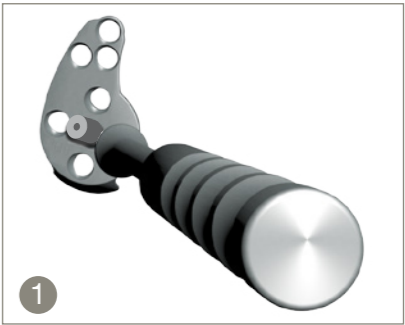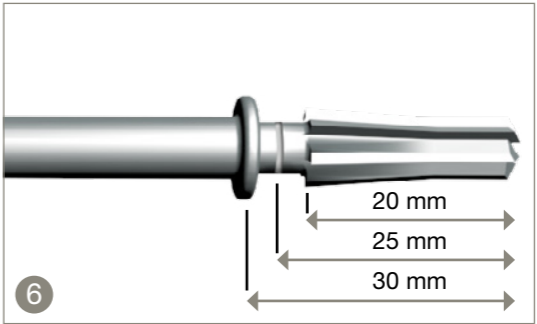

Introduce the back and forth reamer into the conically drilled hole, and ream the glenoid surface with alternative clockwise anticlockwise movements 7.

Assemble the trial revision peg of the length previously determined (20, 25 or 30mm) onto the trial glenoid of the selected side 8.

Test for bone cover and stability 9.

Remove the definitive implant of selected side and length of peg from its packaging.

Maintain the polyethylene morse taper protection in place.

Assemble the implant onto the baseplate impactor (E28 210).




The plate may be shortened by cutting between the most proximal Holes if necessary to adapt to morphology.

Bone graft can be used behind the baseplate or between the peg and inferior lip, dependent on the condition of the bone.


Implant the glenosphere as for the reverse primary shoulder (page 17 of the UNIC Reverse Surgical Technique E28 494).

Complete the fixation with screws.


Note that the diameters of screws are different for the superior plate holes (Ø4.3mm « RS » references screws) and for the base plate (Ø5.0mm).

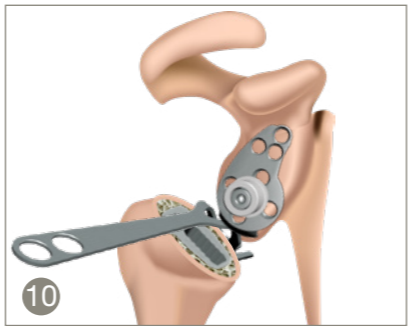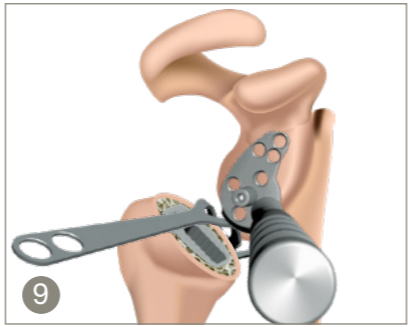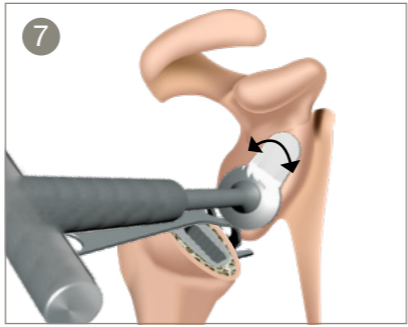


E28 9105 UNIC Reverse

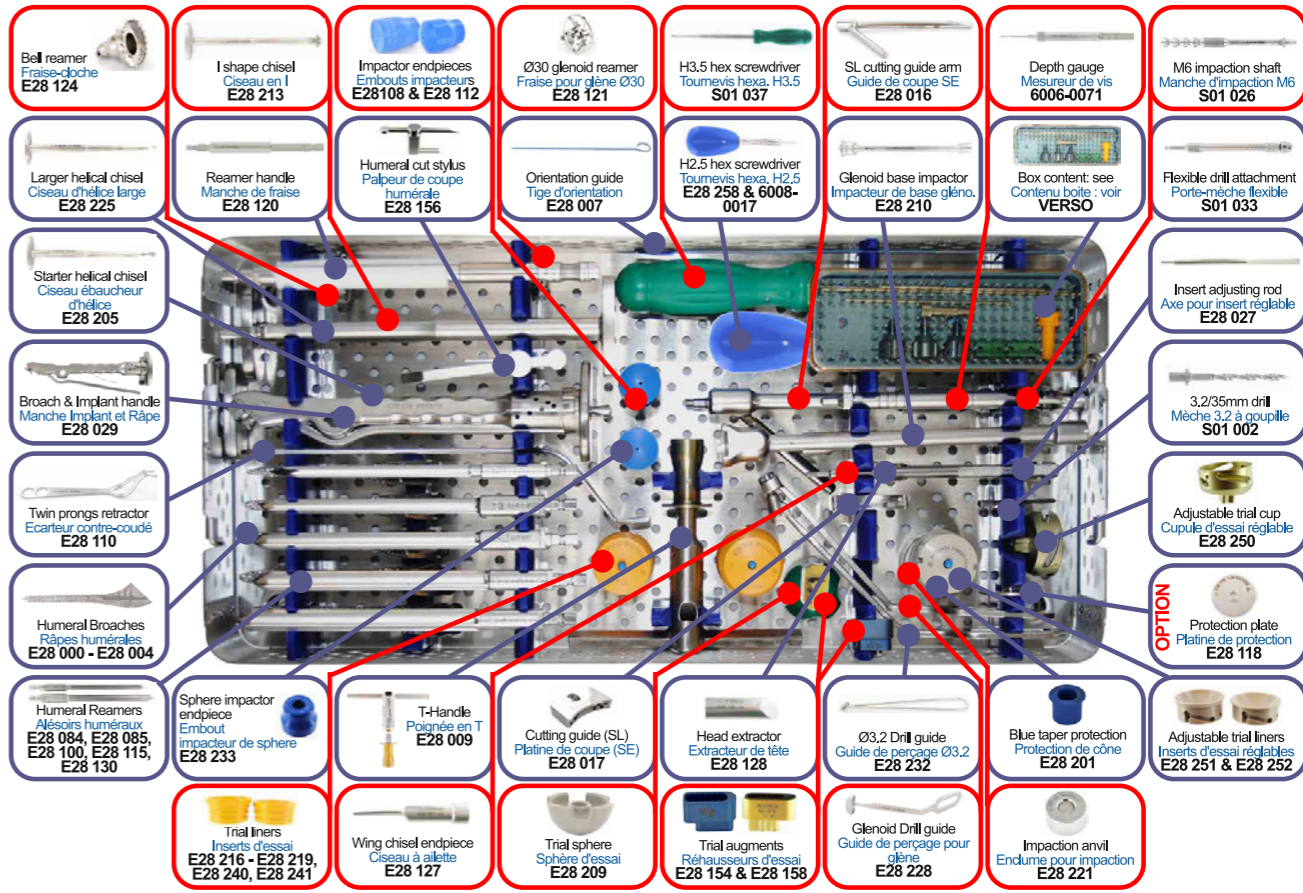


E28 9106 UNIC Anatomic and CTA

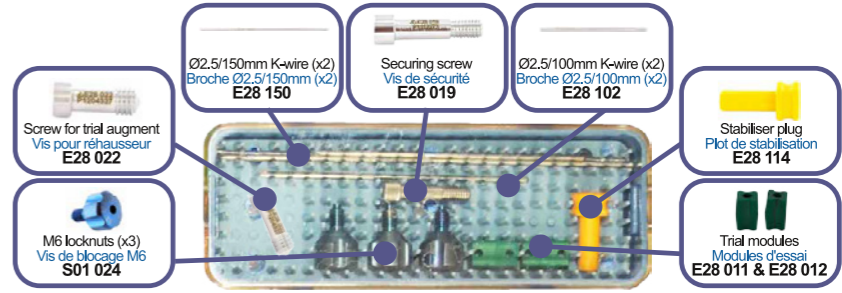

E28 9102 UNIC Humeral Revision

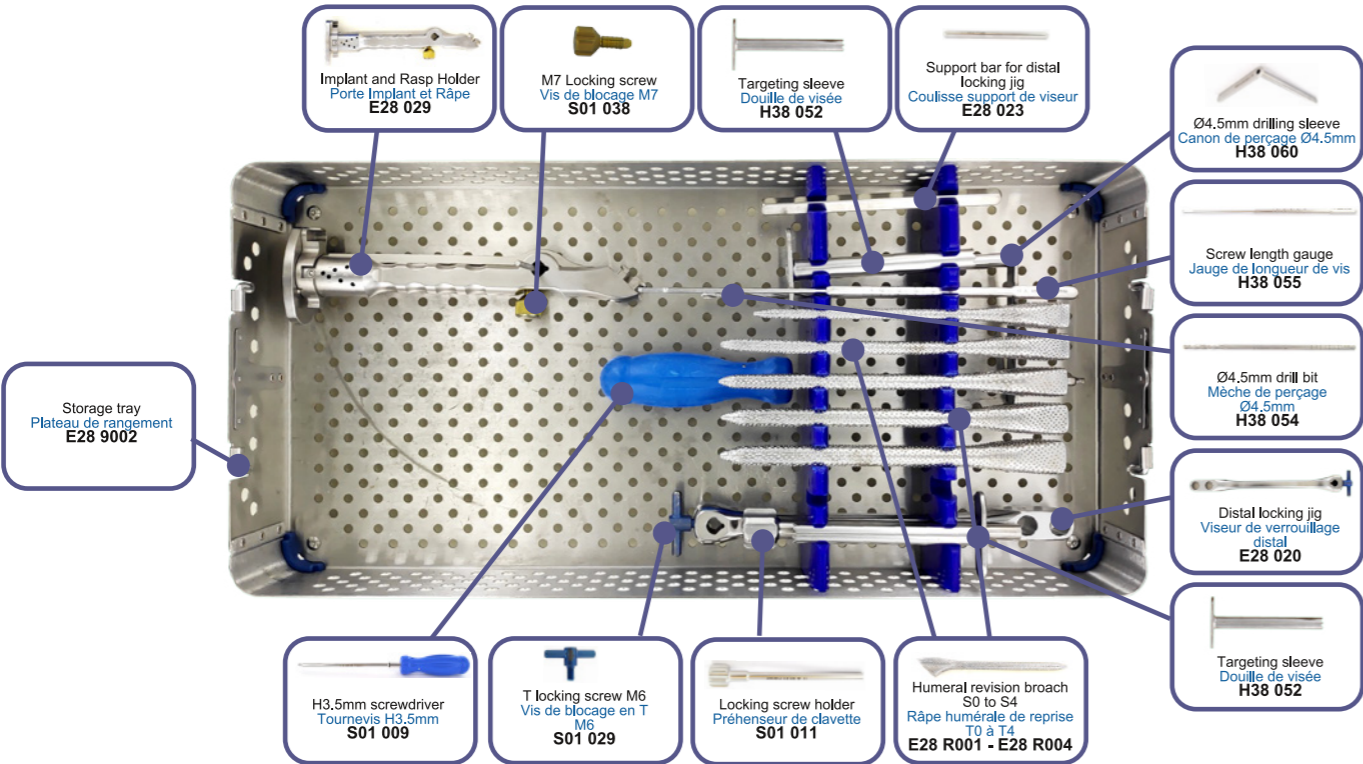

E28 9103 UNIC Glenoid Revision

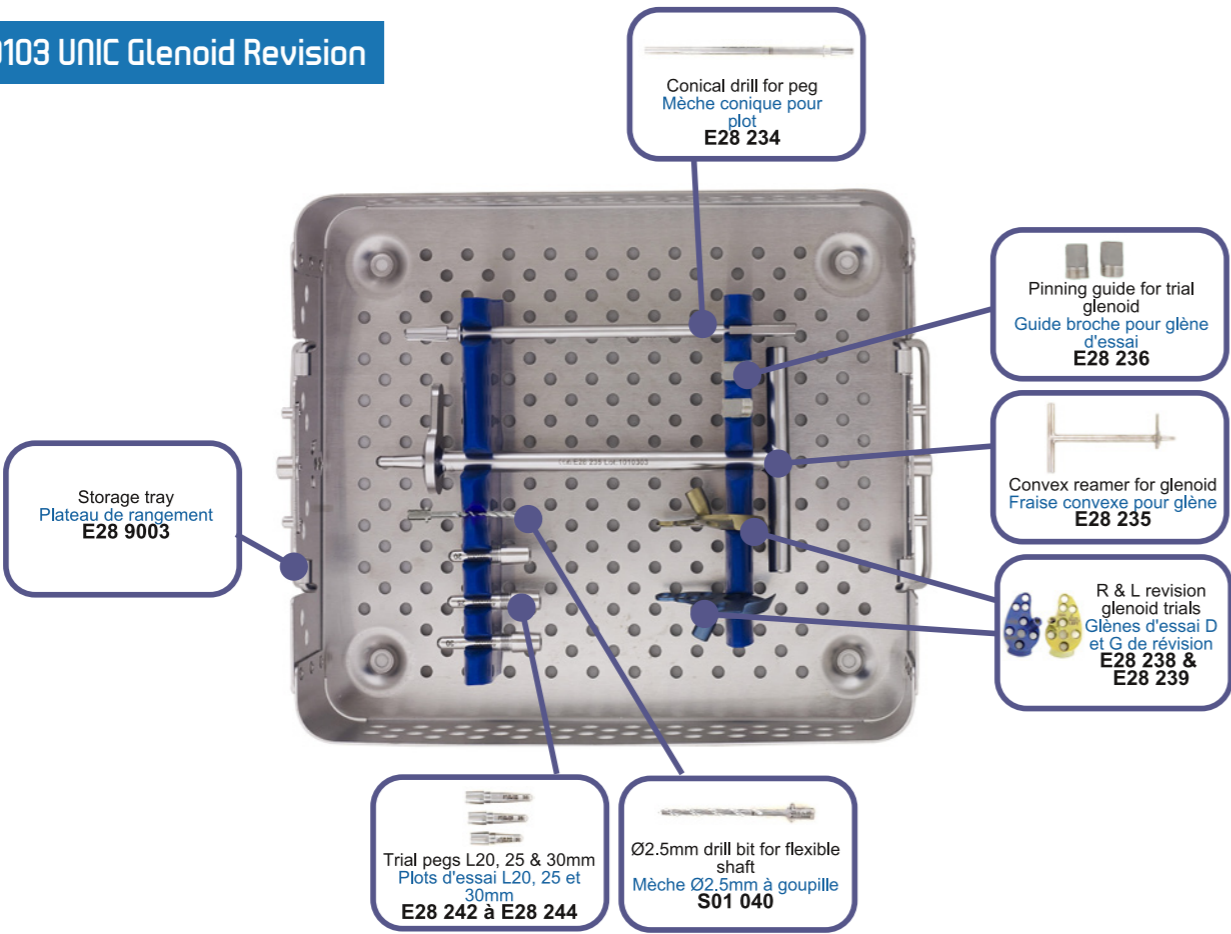

Reverse Shoulder / Epaule Inversée**Revision Glenoid Baseplate / Embase Glénoïdienne Révision**

Side / Côté	L. (mm)	Reference
Right / Droit	20	E27 200RD1
	25	E27 200RD2
	30	E27 200RD3 ⁽⁴⁾
Left / Gauche	20	E27 200RG1
	25	E27 200RG2
	30	E27 200RG3 ⁽⁴⁾

Screw for glenoid base / Vis pour base glénoïdienne

Description	Ø (mm)	L. (mm)	Reference
For / Pour XS Ø26 and / et Revision	4.3	15	E27 415RS ⁽¹⁾⁽³⁾
		17.5	E27 417RS ⁽¹⁾⁽³⁾
		20	E27 420RS ⁽¹⁾⁽³⁾
		22.5	E27 422RS ⁽¹⁾⁽³⁾
		25	E27 425RS ⁽¹⁾⁽³⁾
		30	E27 430RS ⁽¹⁾⁽³⁾
		35	E27 435RS ⁽¹⁾⁽³⁾
For / Pour Standard Ø30	5.0	40	E27 440RS ⁽¹⁾⁽³⁾
		15	E27 515
		20	E27 520
		25	E27 525
		30	E27 530
		35	E27 535

Humeral stem / Tige humérale**Revision stem / Tige de Révision**

Size / Taille	Cemented Cimentée	L. (mm)	Cementless Sans ciment
S.00	E27 R020XS ⁽⁴⁾	179	E27 R000XS ⁽⁴⁾
S.0	E27 R020 ⁽²⁾	180	E27 R000 ⁽²⁾
S.1	E27 R021 ⁽²⁾	200	E27 R101 ⁽²⁾
S.2	E27 R022 ⁽²⁾	200	E27 R102 ⁽²⁾
S.3	E27 R023 ⁽²⁾	200	E27 R103 ⁽²⁾
S.4	E27 R024 ⁽²⁾	200	E27 R104 ⁽²⁾

Locking screw for Revision stem / Vis de verrouillage pour tige de Révision

Description	Ø (mm)	Reference
Ø6.0 / Ø4.5	20	H15 SC6020 ⁽²⁾
	25	H15 SC6025 ⁽²⁾
	30	H15 SC6030 ⁽²⁾
	35	H15 SC6035 ⁽²⁾
	40	H15 SC6040 ⁽²⁾

(1) Product included in the XS complementary set, not supplied with the standard basic set.

(2) Product included in the humeral revision complementary set, not supplied with the standard basic set.

(3) Product included in the glenoid revision complementary set, not supplied with the standard basic set.

(4) Product available only on request. Not provided in standard order sets.

Please note: the references mentioned above are those specific to revision implants. The articular elements (eg sphere + cup or head + glenoid) are presented in the UNIC Reverse (E28 430) and UNIC Anatomic brochures (E28 486) as well as in the UNIC Reverse (E28 494) and UNIC Anatomic surgical techniques (E28 491).



UNIC®
Reverse
revision



Revision glenoid
baseplate and glenoid
sphere

Important Notice:

The UNIC Revision implants belong to the class III implantable medical device classification. The UNIC Revision implants are indicated in reverse total shoulder revision arthroplasty procedures.

The surgeon is required to read the instructions for use (IFU) S12 0314 included in the packaging of the implant or available for download from the www.evolutis-group.com website, as well as the surgical technique manual E28 494 (UNIC Primary), ou E28 497 (UNIC Revision) initially delivered with the instrument set, or equally available for download from the www.evolutis-group.com website.

Materials:

Humeral stem: Titanium alloy (TA6V) according ISO 5832-3, with T40 and calcium hydroxyapatite coatings (cementless) or Shiny polished titanium alloy according ISO 5832-3 (cemented)

Standard & XS Glenoid sphere, security screw for sphere: M30NW stainless steel according ISO 5832-9

Humeral cup, humeral augment, cancellous screw & locking screw: Titanium alloy (TA6V) according ISO 5832-3

Glenoid base (Helical, Long Post & Revision: Titanium alloy (TA6V) according ISO 5832-3, with calcium hydroxyapatite coating

Humeral inserts: UHMWPE according ISO 5834-1 & 2

Packaging: Sterilized under Gamma irradiation, VacUpac packaging